




Tourni-S™ is a single use disposable tourniquet designed for optimum comfort. It is safe and easy to use, its simple construction makes it lightweight, portable and easy to apply.

This tourniquet has been carefully designed using skin friendly materials that do not pinch the skin. The product is latex free and adheres to European health and environment standards. This cost effective product also helps prevent cross infections by being single use, thereby eliminating additional costs to treat these infections.

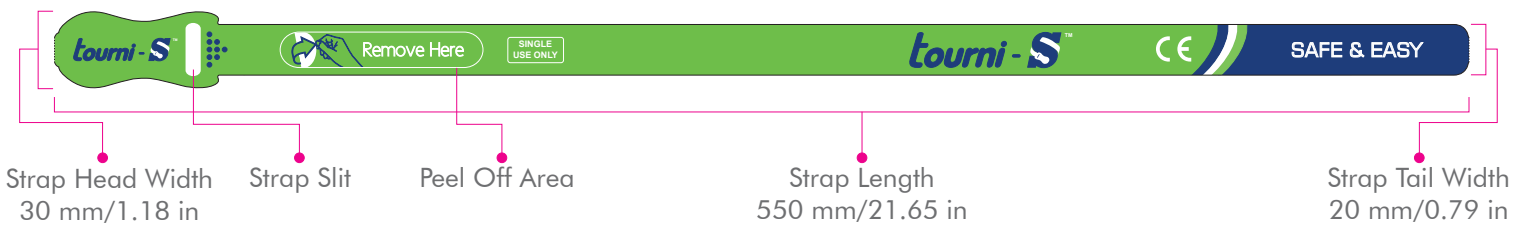

## Instructions for use

- 1 PEEL**  
Peel off adhesive cover labelled 'Remove Here' as indicated on the strap. Do not let the adhesive become contaminated with dust or finger prints.
- 2 WRAP**  
Wrap the tourniquet around the individuals arm with the printed side facing upwards.
- 3 ADJUST**  
Adjust/Re-adjust the strap around the arm by passing the strap tail through the slit on the strap head and adjust according to the arm size.
- 4 FASTEN**  
Once fully adjusted, secure the strap around the adhesive area by applying firm pressure using your thumb.
- 5 REMOVE & DISPOSE**  
Carefully remove the tourniquet after use and dispose in a safe manner. Do not flush, could clog wastewater systems.

## Types of Packs

Tourni-S come in two pack sizes.

### Compact Pack:

This pack consists of 50 tourniquet strips.

### Large Pack:

This pack consists of 200 tourniquet strips.

## Order Information

### TourniS.SK.AD.30550.50

Tourni-S Single use disposable Tourniquet. Adult size 30 x 550mm. Pack of 50 pcs (50 pcs per pack)

### TourniS.SK.AD.30550.200

Tourni-S Single use disposable Tourniquet. Adult size 30 x 550mm. Pack of 200 pcs (200 pcs per pack)