REF: ZIT-500, ZIT-502, ZIT-1000, ZIT-1002, ZIT-3000, ZIT-3002

InfuseIT™ Disposable Infusion Bag

(IFU LV05008 Rev W 2021/10)

EN
DEVICE DESCRIPTION: The device is non-sterile and disposable. A device with an inflatable bladder, a pressure gauge, and an inflation bulb. **INTENDED PURPOSE:** A pressure infusor has an inflatable sleeve which is placed around an IV bag. When the device is inflated, it assists in the infusion of fluid. **INDICATIONS:** Where rapid infusion of fluids is required or for use with invasive blood pressure monitoring. **ENVIRONMENT:** Hospital, sub-acute, pre-hospital **PATIENT TARGET GROUP:** Newborns, infant, pediatric, adult **EXPECTED CLINICAL BENEFITS:** Enables rapid infusion to replace fluid loss. Ensure infusion line patency. **CONTRAINDICATIONS:** None **Known WARNINGS:** None **Known CAUTIONS:** Follow cleaning instructions to avoid risk of infection and contamination **INSTRUCTIONS:** Contact your country’s Competent Authority to report any serious incident. Dispose of device in accordance with local, state or national regulations. Follow instructions for proper care of device to prevent risk of contamination and injury. This product is disposable and is not intended for prolonged use.Maintain proper inflation pressure during use. Ensure green band is displayed on pressure gauge. Do not overpressurize. 1.SELECT the correct size infusor for the IV fluid bag. 2. INSERT IV fluid bag between the pressure infusor and sleeve. 3. HANG IV fluid bag on the infusor’s plastic hook. 4. HANG pressure infusor on IV pole or other location IV fluid bags are normally hung. 5a. Inflation using Thumbwheel Model 1. Turn the thumbwheel clockwise until snug. This is the fully CLOSED position. DO NOT over tighten. 2. SQUEEZE inflation bulb repeatedly until a pressure of 300mmHg (green band)is displayed on pressure gauge. 3. To Deflate: Turn the thumbwheel counter-clockwise to the fully OPEN position. 5b. Inflation using Stopcock Model 1. TURN the stopcock valve so the arrow points to the right. This is the INFLATE position. 2. SQUEEZE the inflation bulb repeatedly until a pressure of 300mmHg (green band) is displayed on pressure gauge. 3. TURN the stopcock valve so the arrow points UP (towards the infusor). This is the HOLD position. The stopcock valve is required to be in this position for as long as pressure is to be held. 4. To Deflate: Turn the stopcock valve so the arrow points down (towards the inflation bulb). **Caution:** Maintain proper inflation pressure during use. Ensure green band is displayed on pressure gauge. Do not overpressurize. **Cleaning Instructions:** InfuseIT manual pressure infusors are disposable devices but may be cleaned by using the following method: Note: Do not soak or immerse the infusor. APPLY either a 10% bleach solution or hospital-approved detergent for porous surfaces to the infusor. USE a soft brush or cleaning cloth to remove stains. RINSE thoroughly with running water. Ensure the vent hole in the pressure gauge is facing down so water doesn’t enter the gauge. ALLOW infusor to air dry before use.Discard and replace immediately if the infuseIT becomes damaged and fails to maintain pressure, is unable to be cleaned or has been contaminated with blood or other bodily fluids. Gauge accuracy is ±10%.

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