


Split sheet w/ Drisite® Plus for abdominal surgery

- BARRIER® split sheets are purpose-designed featuring only impermeable materials to minimise the risk of strikethrough in all surgical situations
- Reinforced materials in the critical zone minimise tearing and risk of breach in the infection control system
- The patented Drisite Plus system enhances drape absorption to minimise risk of bacteria transfer and unwanted liquids on the OR floor
- The new BARRIER surgical drapes feature soft and comfortable materials that conform naturally to the body. This assures adhesion and minimises risk of leakage

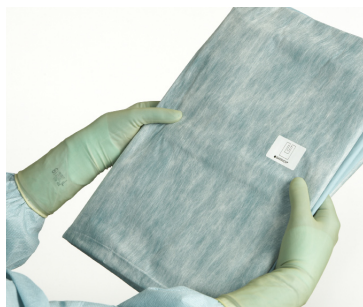



1 Position the split sheet towards the patient as indicated by instruction label.



2 Position the split sheet in the middle of the abdominal area.



3 Unfold the drape side to side.



4 Unfold the drape towards patient's feet.



5 Remove the release paper.



6 Secure adhesive edge one side at a time.



7 Position the adhesive op-sheet in the middle.



8 Unfold the drape.

| Sterile codes | |
|--------------------------|--|
| 60503 | Split sheet 200×260 cm, split 7×102 cm |
| 60504 | Split sheet 200×260 cm, split 20×102 cm |
| 60506 | Split sheet 200×290 cm, split 7×102 cm |
| 60502 | Split sheet 200×150 cm, split 7×65 cm |
| ProcedurePak® Components | |
| 70503 | BNS Split sheet 200×260 cm, split 7×102 cm |
| 70504 | BNS Split sheet 200×260 cm, split 20×102 cm |
| 70506 | BNS Split sheet 200×290 cm, split 7×102 cm |
| 70502 | BNS Split sheet 200×150 cm, split 7×65 cm |
| 70508 | BNS Split Sheet 230×150 cm, split 7×65 cm, patched |
| 70509 | BNS Split Sheet 150×260 cm, split 7×65 cm, patched |
| 70511 | BNS Split sheet 230×330 cm, split 20×102 cm |
| 76409 | OP-sheet 240×175 cm, patched |
| 76410 | OP-sheet 300×175 cm, patched |

