



Nitrile gloves, powder-free

Art. 290423, 3958-3961, 3945

Chemical	CAS number	Permeation time	Class
1,2 dichloroethana		<1 min.	0
2-Hydroxyethyl methacrylate 97%		10 min.	1
0.5 M iodine solution	7553-56-2	>480 min.	6
2-Mercaptoethanole	60-24-2	3 min.	0
Acetic Acid 10%		241 min.	5
Acetonitrile >99%	75-05-8	<1 min.	0
Acrylamide 40%	79-06-1	>480 min.	6
Ammonia 25%	1336-32-6	13 min.	1
Chlorine 1.000 ppm		>480 min.	6
Chlorhexidine 4%	55-56-1	>480 min.	6
Chlorhexidine Gluconate 4%	18472-51-0	>480 min.	6
Citric Acid 15%		>480 min.	6
Cyclohexanol	108-93-0	68 min.	3
Dichloromethane >99%	75-09-2	<1 min.	0
Diethylamine	109-87-7	<1 min.	0
Diurethane dimethacrylate		320 min.	5
Ethanol 70 %	64-17-5	87 min.	3
Ethanol 80 %	64-17-5	15 min.	1
Ethidium bromide 1%	1239-45-5	>480 min.	6
Ethyl Acetate >99%	141-78-6	<1 min.	0
Formamide 40%	75-12-7	>480 min.	6
Formaldehyde 37% in Methanol 10%	50-00-0	>120 min.	3
Formaldehyde/Formalin 37%	50-00-0	>480 min.	6
Formic Acid 12,5%	64-18-6	240 min.	4
n-Hexane	110-54-3	>480 min.	6
Hydrochloric acid 36%	7647-01-0	140 min.	4
Hydrofluoric acid 40%	7664-39-3	6 min.	0
Hydrogen peroxide 3%	7722-84-1	>480 min.	6
Hydrogen peroxide 30%	7722-84-1	31-45 min.	2
Isopropanol 45%	67-63-0	>480 min.	6
Methanol 5%	67-56-1	>480 min.	6
Methanol 10%	67-56-1	30 min.	1
Methyl methacrylate		< 1 min.	0
Nitric acid 10%	7697-37-2	>480 min.	6
n-heptane	142-82-5	80 min.	3
n-Propanol 60%		51 min.	2
Peracetic acid 38%	79-21-0	9 min.	0
Phosphoric acid 30%	7664-38-2	>480 min.	6
Perchloric acid 70 %		92 min.	3
Petroleum – distilled		5 min.	0
Phenol 90%		9 min.	0
Potassium hydroxide 30%	1310-58-3	>480 min.	6



Nitrile gloves, powder-free

Art. 290423, 3958-3961, 3945

Povidone iodine 10%	25655-41-8	>480 min.	6
Rely+On™ Virkon® (Solution of potassium Peroxymonosulphate)	70693-62-8	>480 min.	6
Silver nitrate 10%	7761-88-8	>480 min.	6
Sodium carbonate 5%		>480 min.	6
Sodium hydroxide 40%	1310-73-2	>480 min.	6
Sulphuric acid 50%	7664-93-9	>480 min.	6
Sulphuric acid 96%	7664-93-9	8 min.	0
Tri(ethylene glycol) dimethacrylate 95%		33 min.	1
O-Xylene >99%	95-47-6	<1 min.	0

Cytostatica

Flourouracil – 50.000 ppm		>240 min.	5
Carmustine 3.300 ppm		35 min.	2
Cisplatin 1.000 ppm		>240 min.	5
Cyclophosphamide 20.000 ppm		>240 min.	5
Dacarbazine 10.000 ppm		>240 min.	5
Doxorubicin Hydrochloride 2.000 ppm		>240 min.	5
Etoposide 20.000 ppm		>240 min.	5
Methotrexate 25.000 ppm		>240 min.	5
Mitomycin C 500 ppm		>240 min.	5
Paclitaxel (taxol) 6.000 ppm		>240 min.	5
Thio-Tepa 10.000 ppm		85 min.	3
Vincristine sulfate 1.000 ppm		>240 min.	5

Antibiotica

Cefuroxim 15.000 ppm		>480 min.	6
Dicloxacin 10.000 ppm		>480 min.	6
Gentamicin 1.200 ppm		>480 min.	6

Test method used: EN 374-3:2003, EN16523-1:2015 and ASTM D6978